



Some words to the collection of hair samples

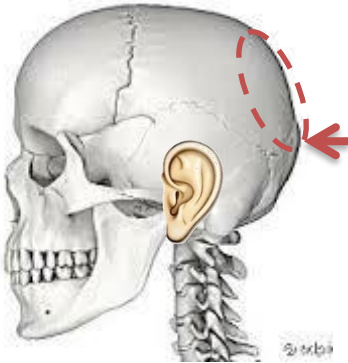


Where to collect the hair sample

- Hair samples collected at the upper back of the head („**posterior vertex**“ = between highest point of the head/vertex and bump of the back of the head:
For orientation, slightly above the height at the tip of the ear
→ see Figure left

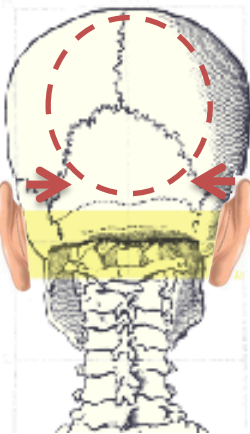

How much amount of hair is needed?

- several (**2-3**) small strands at the back of the head as close as possible to the scalp (please use the loops)
→ overall diameter: **half a pencil** (see below)

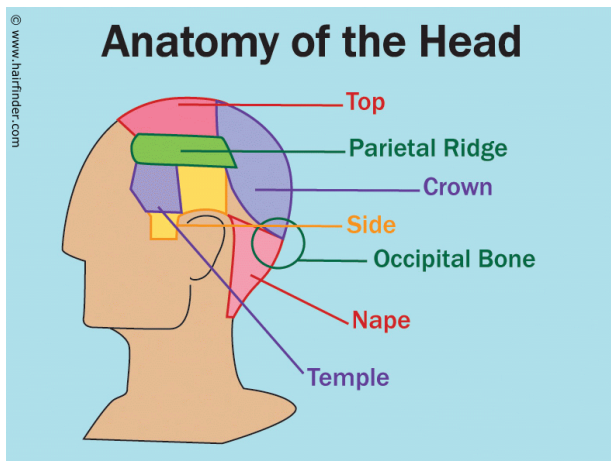

Where to collect the hair samples

- Hair samples collected at the upper back of the head („**posterior vertex**“ = between highest point of the head/vertex and bump of the back of the head
→ see Figure left: „Crown“ (violett); for optical reasons, the lower part of this region is recommended so that the top layer of hair may cover possible visible points of sampling

For orientation,

The **overall diameter of the hair sample** (i.e., all small samples taken altogether) corresponds with the size of half of a pencil:

